

## Software-Update plusoptiX S09

1. Prepare an empty USB flash drive with a minimum capacity of 2GB.
2. Click on the download-link with the newest software update which you can find on our homepage and save the file on your computer.
3. Unpack the downloaded file onto your empty USB drive.
4. To make your Update bootable start the "bootable.bat" file in the directory of the USB flash drive.
5. Your USB drive is now ready to perform software updates.
6. Switch off the plusoptiX S09.
7. If a printer is connected to the plusoptiX S09 via an USB-cable, please disconnect the USB-cable form the plusoptiX S09.
8. Please insert the attached USB drive with label and connectors facing up in a USB port of the plusoptiX S09.
9. Switch on the plusoptiX S09.
10. Please wait until this window appears on the monitor:

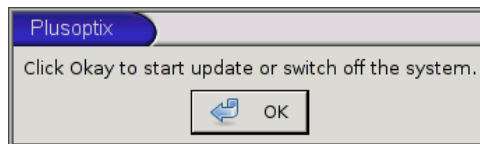


Figure 1: Click OK to start Update

11. Please click on "OK" to start the software update.
12. Thereafter a window appears indicating that the software update is in progress:

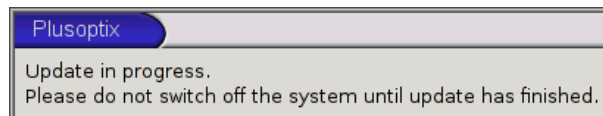


Figure 2: Update in progress

13. The software update may last up to 15 minutes. Please do not switch off the plusoptiX S09 during the update.
14. As soon as the software update is completed this window appears on the monitor:

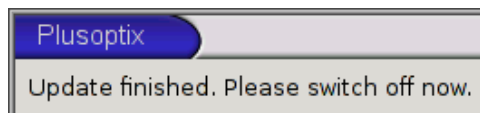


Figure 3: Update finished

15. Switch off the plusoptiX S09 and pull out the USB drive.
16. Please connect now the USB-cable for the printer again, if available.
17. Switch on the plusoptiX S09 again.
18. Click on „Settings” → „System”. → „C) Keyboard“ change it e.g. to „USA”.
19. Afterwards click on „binocular” on the top left in order to continue measurements.